

PATIENT INFORMATION CARD

- Approximate treatment time for 40mAmp* dosage is 1 hour and 15 minutes. Approximate treatment time for 80mAmp* dosage is 2 hours and 30 minutes.
- Remove ActivaPatch® after LED turns off **or** after 4 hours.
- DO NOT get ActivaPatch® wet.
- DO NOT engage in aggressive activity while wearing the ActivaPatch®.

LED Flash Sequence

One Flash	per 4 seconds	Current is not flowing. Ensure patch is properly adhered to skin.
Three Flashes	per 10 seconds	Minimal current flow; timer has not begun. Leave patch in place for 10 minutes or until one flash occurs.
One Flash	per 30 seconds	Normal current flow; timer will begin. See Dosage Timer Indications for treatment sequence.

Dosage Timer Indications

One Flash	per 30 seconds	First dosage in process.
Three Flashes	per 10 seconds	3-minute transition to second dosage period. (3 short/long/short flashes) PER 10 seconds FOR three minutes
Two Flashes	per 30 seconds	Second dosage in process.
Off		End of treatment. Remove ActivaPatch®.